



General Device Information: Changing Logout Times on a Device



Contents

Purpose 1

Change Logout Time 1

Purpose

This document provides guidance on how to change the automatic logout time on the tablet of the device. If the tablet is not used within this timeframe, it automatically logs out the user.

Change Logout Time

- 1 | Log into the tablet using an account with administrative privileges.
 - a. If your account does not have administrative privileges, login using the System Administrator or Training User account (password: optomap).
- 2 | Select **To admin**.
- 3 | Select **System Preferences**.
- 4 | Change the automatic logout time from the dropdown menu.
- 5 | Select **Save these settings** to save.

