



General Device Information System Reboot



Contents

Purpose 1

System Reboot 1

Purpose

This document will provide guidance on the process to properly reboot your Optos® device.

A reboot of the device should be performed once per week with a preference for Wednesday. Always ensure to follow the process outlined in this document.

System Reboot

1 | From the device tablet, press the **Shut Down** button in the top left corner.



2 | Turn off the device by pressing the on/off switch on the back of the device.

3 | Reboot the PC by clicking on the **Windows icon** on the bottom left of the screen, selecting Power Options, and then **Restart**.



4 | When the PC has restarted, log in using your credentials:

5 | When the PC has fully logged in, turn the device back on by pressing the on/off switch.

