



Optos*Advance*TM Accessing Review Software



Contents

Purpose	1
Local Optos <i>Advance</i>	1
Using an IP Address	1
Optos <i>Cloud</i> TM	2

Purpose

This document provides guidance on how to access Optos*Advance* review software on computers connected to the same network as the Optos PC.

Local Optos*Advance*


Open an Internet browser window such as Google Chrome or Microsoft Edge. Our software performs best with Chrome.

- 1 | Type `http://optosadvance` in the address bar and press enter.
- 2 | The login screen will appear.
- 3 | You may add a shortcut to your desktop or taskbar once at the login screen.

Using an IP Address

Depending on your network set up, you may need to use an IP address from the Optos PC to access the review software.

- 1 | Login into the Optos PC.
- 2 | Click on the **search box** in the bottom of the screen and type `CMD`, then press enter.

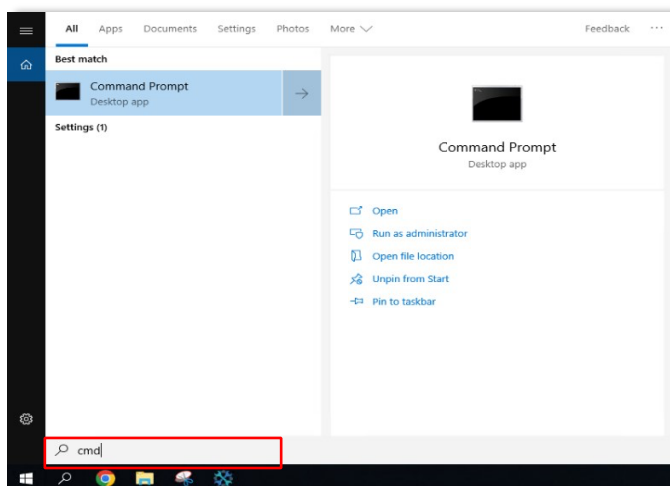





Optos*Advance*TM Accessing Review Software



3 | From the Command Prompt window, type "ipconfig" and press enter.

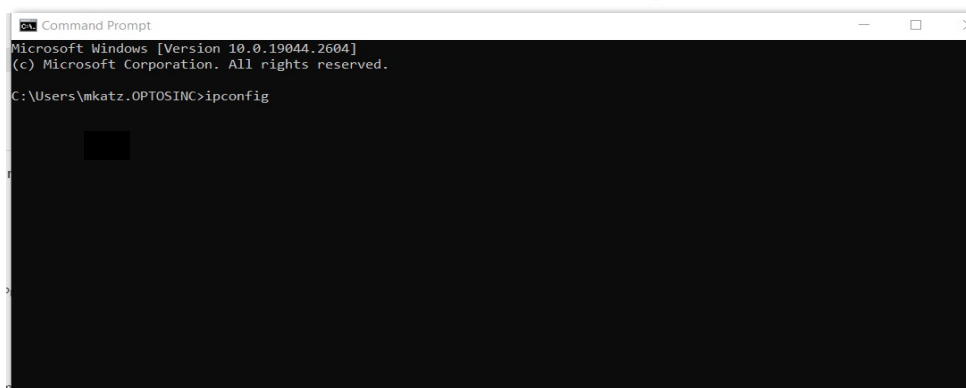

4 | Find the IP address for the network adapter and make a note of the address listed.

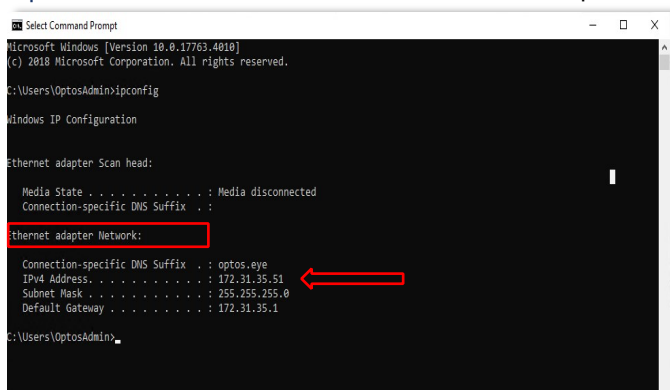

5 | Go to the computer to add review software and input the IP address into browser window address bar. (In this example, the IP address is 172.31.35.51)

6 | You may add a shortcut to your desktop or taskbar.

Optos*Cloud*

Open an Internet browser window such as Google Chrome or Microsoft Edge. Our software performs best with Chrome.

1 | Access Optos*Cloud* using the appropriate country-specific link below:

Optos <i>Cloud</i> US	https://cloud.optos.com/
Optos <i>Cloud</i> Canada	https://ca.cloud.optos.com/
Optos <i>Cloud</i> Europe	https://eu.cloud.optos.com/
Optos <i>Cloud</i> Australia	https://au.cloud.optos.com/

