



Optos *Advance*™ How to Print Images



Contents

Purpose 1

How to Print Images..... 1

Purpose

This document provides guidance on how to print images in *OptosAdvance*.

How to Print Images

You can print the patient study currently loaded in the image viewing area, print one image, or print all viewports.


- 1 | If you want to print a single viewport, select the desired viewport before proceeding.
- 2 | Select **Print** from the toolbar.



- 3 | Select from the options provided:
 - **Save** as one of the following:
 - Viewport - the window selected prior to selecting the Print option
 - Display - the image(s) that are on the screen
 - Current Image - the image that is selected
 - Current Series - all images of one modality type
 - Current Study - all images from the study
 - Key Images
 - Select the preferred **Resolution** 100%, 200%, 300%, 400%.
 - **Template** - Do not modify; ensure **Custom** is selected.
 - Choose **Titles** option appropriate for your use. This option will keep/remove information (Name, ID, DOB, etc.) from the image(s) depending on the selection.
 - All, Minimal, Light or None
 - A timed sequence (e.g., *fa* or *icg*) will require 'All' selected



- 4 | At bottom left of the screen Select **Print**.

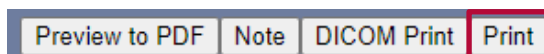

- 5 | When done, select **Close** at the top right of the screen.

