



Optos *Advance*TM Merge Studies



Contents

Purpose	1
Merge Studies	1


Purpose

The document will provide guidance on how to combine two studies into a single study. You may need to perform this action if you imaged the same patient twice in one day.

Users must have the Edit Patient Study privilege in Optos*Advance* to merge studies.

Merge Studies

- 1 | Login to Optos*Advance*.
- 2 | Select the **checkbox** beside the two studies you want to merge.
- 3 | **Right-click** one of the studies, then select **Merge Studies**.

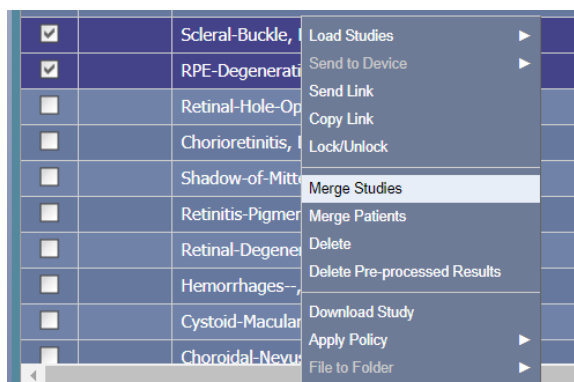

- 4 | Information for the selected studies is shown.



Optos *Advance*TM Merge Studies



Merge Studies		
Attribute	Value	Value
Last Name	Scleral-Buckle	RPE-Degeneration
First Name	Daytona-1	Daytona-1
Middle Name		
Patient Id	1037-D1	1035-D1
Issuer Of Patient Id		
Gender	F ▼	F ▼
DOB	19700101	19700101
Study Id		
Performed On	20151108	20151108
Accession Number		
Referring Physician	Optos^Library^^^	Optos^Library^^^
Description		

Merge to left | Merge to right | **Cancel**

- 5 | To merge the studies into the study shown on the left, select **Merge to left**.
- 6 | To merge the studies into the study shown on the right, select **Merge to right**.
- 7 | Select **Continue** to finalize combining studies into one single study.

