



Optos *Advance*™ Uploading a DICOM Image



Contents

Purpose 1


Upload a DICOM File to Optos *Advance* 1

Purpose


This document will provide guidance on how to upload a DICOM Image to Optos *Advance*.

Upload a DICOM File in Optos *Advance*


- 1 | Open Optos *Advance*.
- 2 | Select Upload New Study on the toolbar.

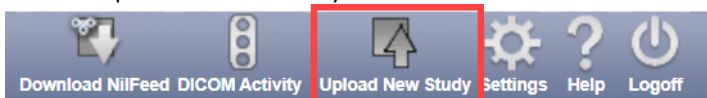


- 3 | Select **Upload Zip with DICOM files**.

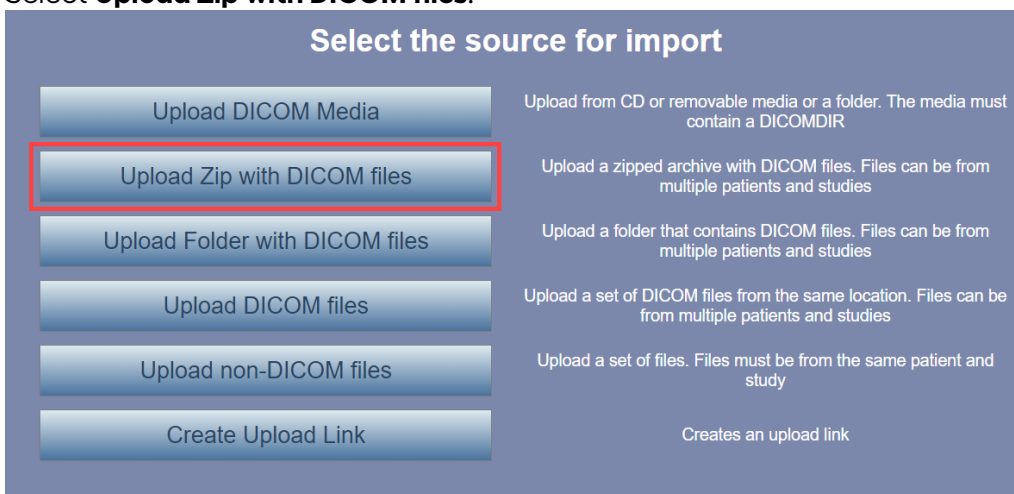

- 4 | Select **Choose File**.

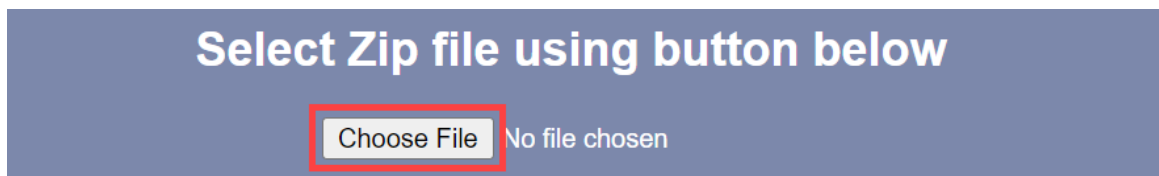

- 5 | Locate the file you want to upload and select **Open**.

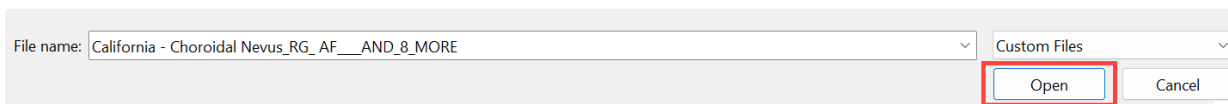



Optos *Advance*TM Uploading a DICOM Image



6 | Select **Upload**

Optos *ADVANCE*TM

Select Zip file using button below

Choose File California - ...8_MORE.zip

Total files to upload: 1 (300.3 MB)

Back Upload Cancel

7 | Once file has been Uploaded and Imported Successfully select **Close**

Back Close Cancel

