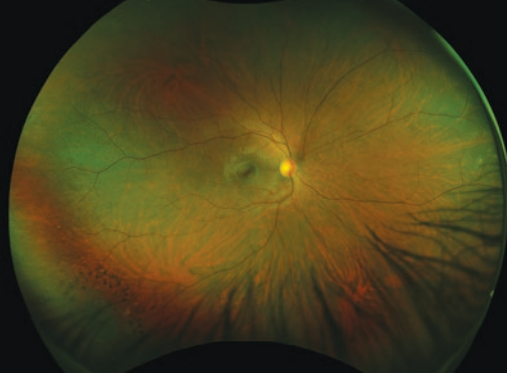
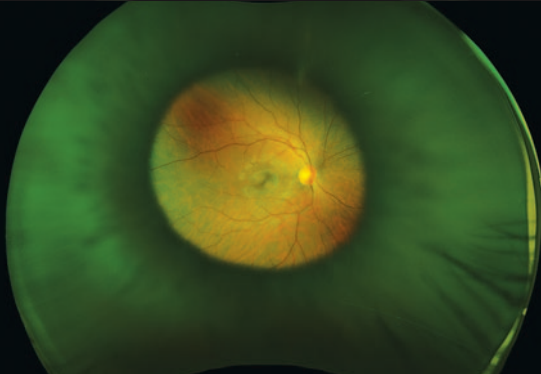
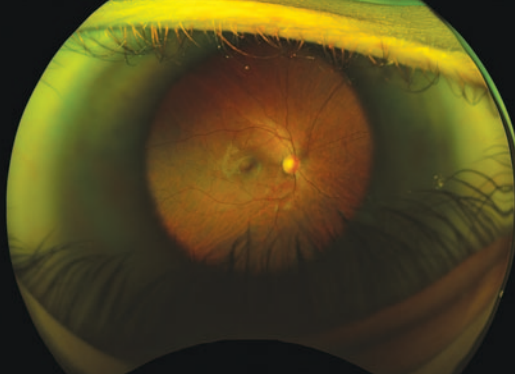
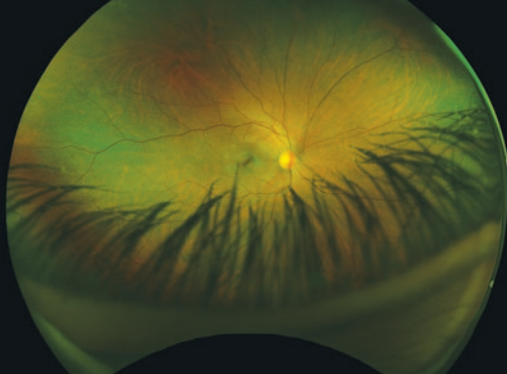
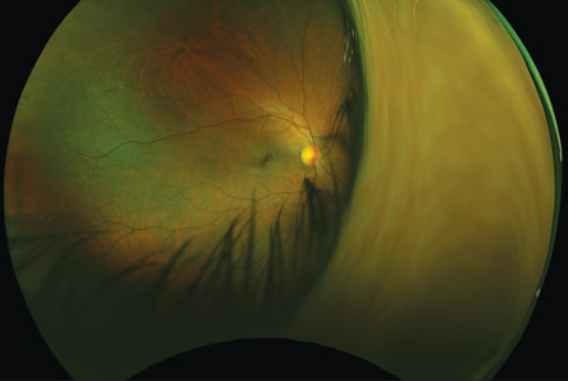
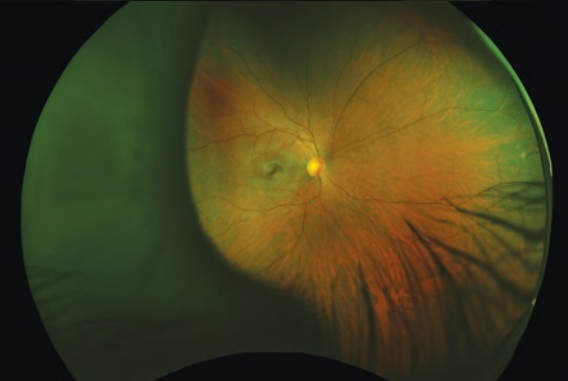
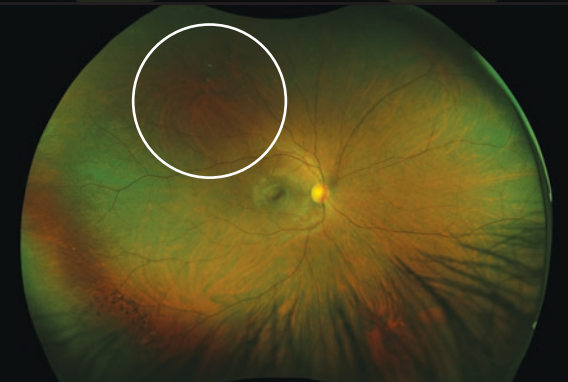
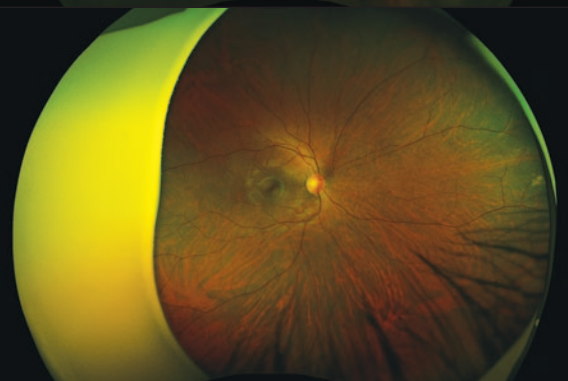
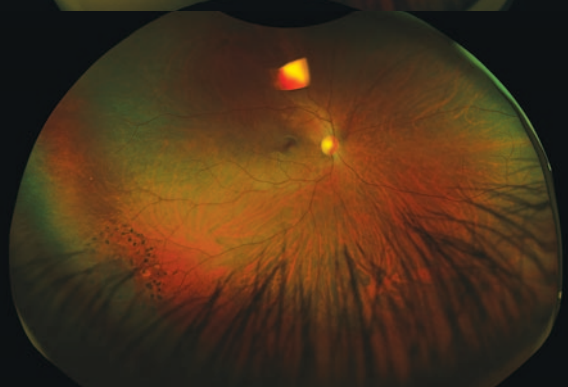


<p>Correct Positioning</p>		<p>This is a good image.</p>
<p>Too Far Out</p>		<p>Patient is too far out, the target is blue.</p> <p>Position patient until target turns green.</p>
<p>Too Close</p>		<p>Patient is too close, the target is red.</p> <p>Position patient until target turns green.</p>
<p>Too Much Eyelid</p>		<p>Have the patient open both eyes as wide as possible. The height of the table may also need to be raised up.</p>

Blink		Have the patient keep both eyes open as wide as possible.
Movement		Patient moved during imaging or the pupil is not centered. Position the patient so the target is in the center of the pupil.
Dust		Dust has accumulated on the main mirror. Clean mirror per protocol.
Facepad		Patient nose is pressed against facepad. Turn the patient's head slightly to the side.
Reflex		Patient is too close or not centered. Reimage.